




synergEyes® PS
post-surgical hybrid contact lens

Important Tips about Wearing SynergEyes® PS Lenses

- » **Give your eyes time to adapt:** Every patient's eyes are different. Because the design of SynergEyes® lenses is so different from other lenses you've tried, there may be a period of adaptation. Your eye care practitioner can explain what the adjustment period may be for you.
- » **Most patients adapt to their lenses in 2-5 days:** Build up your wearing time - start with 4 hours and gradually increase to a full day.
- » **Insertion and removal takes practice:** Review the DVD in your Patient Care Kit or watch on www.synergeyes.com
- » **Be sure to clean your lenses daily:** SynergEyes lenses should be digitally cleaned every day to maintain comfortable, clear vision.
- » **Recommended replacement:** Every six months or more frequently as prescribed by your eye care professional.

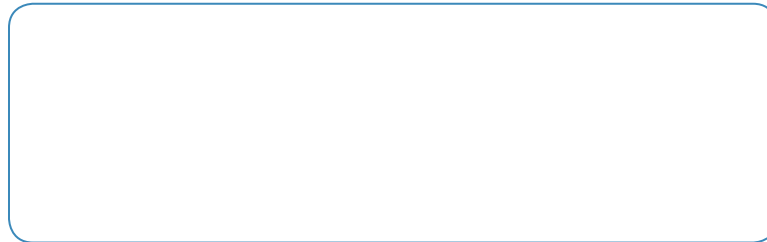

For more information
please visit www.synergeyes.com

A New Found Vision™



synergEyes® PS
post-surgical hybrid contact lens
Patient Information


"I was given the opportunity to try the new *SynergEyes*[®] *PS* lens...all I can say is what a difference! As for vision quality, this lens has definitely offered me the best I've experienced so far."
(Patient, Gabriel M.)



"Once I came across the *SynergEyes*[®] *PS* lenses, my whole world opened up, I was able to see and read and my vision was just amazing. These lenses gave me my life back."
(Patient, Mary Jo P.)

About *SynergEyes*[®] *PS*

If you need vision correction after having refractive surgery, you want correction that provides the excellent optics you expect and the comfort you deserve. *SynergEyes*[®] *PS* hybrid contact lenses combine a rigid gas permeable center with a soft lens outer skirt. This combination provides all-day comfort keeping the lens stable on your eye for consistent, predictable clear vision.

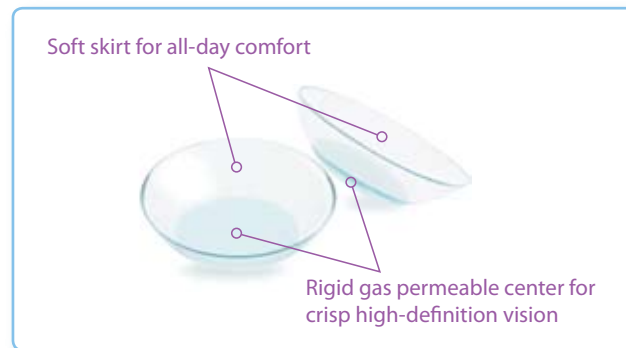

Are you tired and frustrated with disappointing vision correction options?


- Soft contact lenses take the shape of the irregular cornea and may only provide fair vision.
- Thicker soft contacts can reduce oxygen flow to your eye causing edema, an unhealthy "swelling" of the eye.
- Rigid gas permeable or "Hard" lenses can move too much on your eye and can be extremely uncomfortable if dust and debris are caught underneath.

***SynergEyes*[®] *PS* hybrid contact lenses offer you features and benefits never before available in a contact lens.**

The *SynergEyes*[®] *PS* Difference

Each *SynergEyes*[®] *PS* hybrid contact lens is created using patented *SynergEyes* hybrid technology to deliver you the most advanced optics available on the market today. No other contact lens can offer you the unique benefits of *SynergEyes*[®] *PS*.

Your practitioner will custom fit *SynergEyes*[®] *PS* lenses to provide a solution for your individual needs. The soft skirt keeps the rigid portion centered on your eye and allows a layer of tears to form between your eye and the lens, correcting for the irregularities that cause blurry vision, ghosting and halos. And the "breathable" rigid center keeps your eyes healthy while providing crisp, clear vision... even at night!

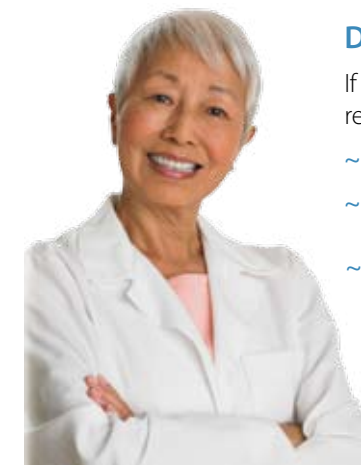

Discover a New Found Vision™

If you need further correction post-refractive surgery, or have experienced:

- ~ Pellucid Marginal Degeneration
- ~ Corneal Trauma/Degenerative Conditions
- ~ Penetrating Keratoplasty and/or Intacs[®] for Keratoconus

SynergEyes[®] *PS* may be the answer you've been looking for! Ask your eyecare practitioner today if *SynergEyes*[®] *PS* is right for you.